

ERGOANALYSIS QUICK GUIDE

PART A

MEASUREMENTS

Get ready:

- Fill Participant Questionnaire
- Read ERGOANALYSIS – PARTICIPANT INFORMATION -document
- MVC test protocol
- Ensure that all devices are charged

Measurement order:

- Moisture the sensors and put clothes on
- Connect the devices and start measurement
- Do MVC and Angular tests
- Do worksite measurements
- End the measurement



PART B

ANALYSIS

Download measurements:

- Attach phone & MCells to laptop, create project & participant (using questionnaire information), download data
- Check that all EMG and angle signals are valid
- Define the areas of MVC and Angular test
- Define Angle offset calibration
- Synchronize data in long measurements (>2hrs)
- Define work phases/tasks
- Create Quick Reports to ErgoCloud of selected results
- Create ErgoAnalysis results in ErgoCloud
- Invite your colleagues/clients to view ErgoAnalysis results by pressing the 'publish' button on selected measurement.

